

Ellipse® Respirator Size Guide




Protecting you for life's **best** moments.

GVS-RPB.COM

The GVS Elipse is a tight-fitting negative pressure respirator, offering a high level of protection and lasting comfort through its ergonomic and lightweight construction. With its flexible design, the Elipse molds to the shapes and contours of your face, ensuring a secure and personalized fit.

Because this is a tight-fitting respirator, selecting the correct size is really important to ensure the respirator can provide you with adequate protection. The Elipse is available in two sizes - S/M and M/L. There are two key measurements that determine the correct size of respirator for you - face length and face width. Cross-reference these measurements on the chart below to find your size.

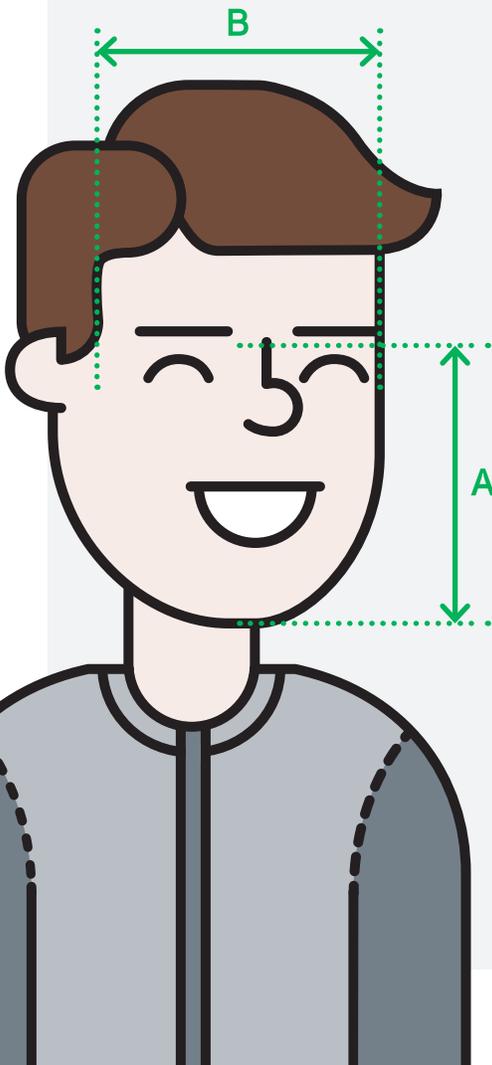

A: Face Length

Measure the distance from the bridge of your nose to the bottom of your chin.

B: Face Width

Measure the distance from one cheekbone to the other.

Face Length (A)	5.1 - 5.5	M/L	M/L	M/L
	4.7 - 5.1	S/M	M/L	M/L
	4.3 - 4.7	S/M	S/M	M/L
	3.9 - 4.3	S/M	S/M	S/M
	4.7 - 5.2	5.2 - 5.7	5.7 - 6.2	
	Face Width (B)			

All measurements are in inches.

Note: This Size Chart is a guide only, and correct sizing and fitment must be qualified using either a quantitative or qualitative face fit test in accordance with national regulations for your region.

For more information contact sales@rpbafety.com